



## INTENDED USE

DermaBind TL can be used as a protective wound covering as either a partial and full thickness acute and chronic wounds.

- Trauma Wounds
- Dehisced Wounds
- Diabetic Foot Ulcers (DFUs)
- Pressure Injuries
- Venous Leg Ulcers (VLUs)


Can be used at the onset and for the duration of the wound, weekly or at the discretion of the treating physician.

## APPLICATION

- Using aseptic technique, remove DermaBind TL from the package; the product may be trimmed as necessary
- Place the DermaBind TL patch onto the wound
- Assure DermaBind TL is in intimate contact with the wound bed; the product can be hydrated once placed on the wound bed
- Adhere DermaBind TL using preferred fixation method, then cover DermaBind TL with a primary non-adherent dressing
- Apply secondary dressings specific to the wound type over the non-adherent layer, which should remain in place for up to one week

## SIZE & STORAGE

- Available in a wide range of shapes and sizes
- Maintain at ambient temperature prior to patient application
- Do not freeze
- May be stored for up to 3 years at room temperature.

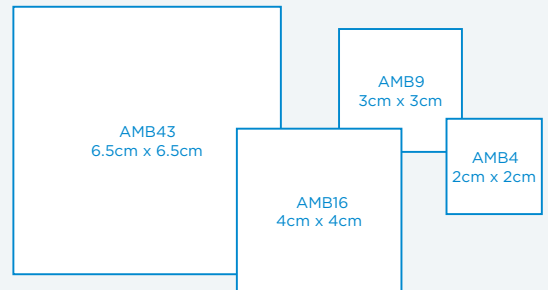

## REIMBURSEMENT

- Medicare contractors cover DermaBind TL for VLUs and DFUs<sup>†</sup>
- DermaBind TL can be applied to, and billed for, other wound types
- Confirm coverage prior to treatment through our Benefits Program

<sup>†</sup>Consult with your LCD (local coverage determination) for the specific wound types covered. VLU = venous leg ulcer; DFU = diabetic foot ulcer

**DermaBind TL  
billing**

**Q 4225**

### CPT codes

Site Preparation CPT<sup>†</sup>  
15002-15005

Application CPT codes<sup>†</sup>  
15271-15278

<sup>†</sup>Subject to guidelines and approvals, please contact our billing team for further details

**BILLING SUPPORT**  
**REIMBURSEMENT@HEALTHTECHWC.COM**

Global Supply Exchange, LLC  
6900 Tavistock Lakes Blvd, Suite 400  
Orlando, Florida 32827

sales@globalsupplyexchange.com  
407-813-2442

Exclusively Distributed By



World Reach Health, LLC

3501 W. Algonquin Rd | Suite 135  
Rolling Meadows, IL 60008